


Flowflex™

SARS-CoV-2

Antigen Rapid Test (Self-Testing)



A rapid test for the detection of SARS-CoV-2 nucleocapsid antigens in anterior nasal swab specimens.
For in vitro diagnostic use only. For self-testing.

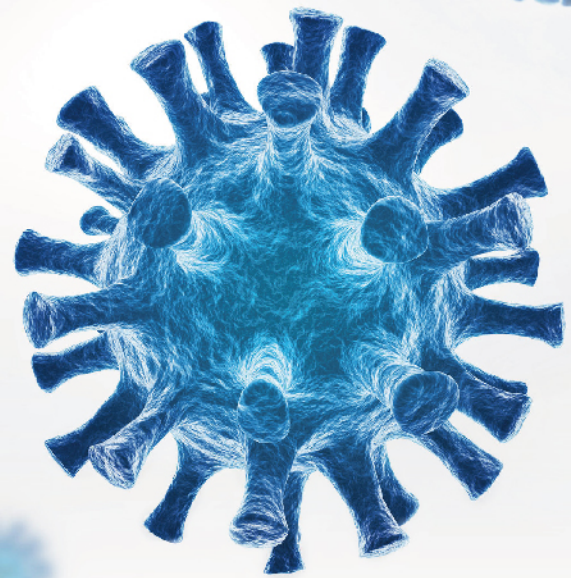

CE 0123

The Global Leader of Rapid Test with **26** Years Experience.



Fast



Accurate



Easy to Use



Reliable



CE Marked

ACON®

Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)

The SARS-CoV-2 Antigen Rapid Test is a lateral flow test for the qualitative detection of the nucleocapsid antigen from SARS-CoV-2 in anterior nasal swab specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- Specimen: Anterior nasal swab specimens
- Test Time: Results at 15 min.
- Shelf life: 24 months
- Storage temperature: 2-30°C
- Accuracy: 98.8%
- Sensitivity: 97.1%
- Specificity: 99.5%



Test Procedure


1. Test Preparation

A. Open your test kit:




B. You should have:

Note: For 1T and 5T, the hole is on the kit box.

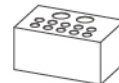

Tube Holder
(for 25T only)

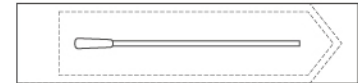

Disposable Swab



Test cassette



Extraction Buffer Tube



Package Insert




Waste Bag


C. Wash or sanitize your hands.



D. Read the instructions.

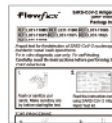

E. Check the expiration date.

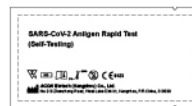

F. Open the pouch.



G. Check the cassette



Result window

Specimen well

2. Specimen Collection


COLLECTION BY AN ADULT CAREGIVER




SELF COLLECTION




or

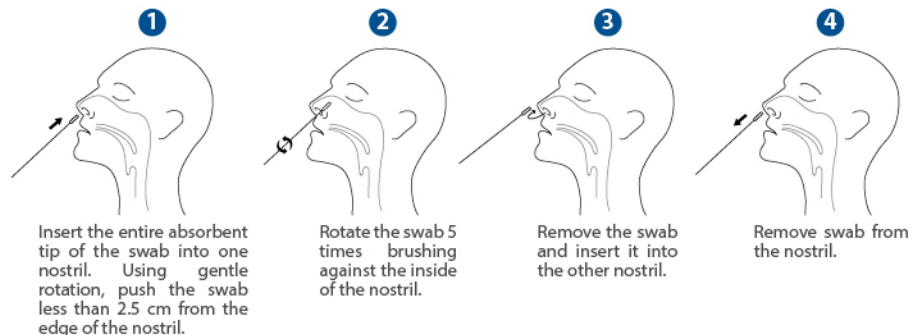


3. Sample Preparation

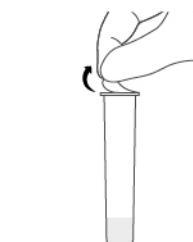


Remove the aluminum foil.

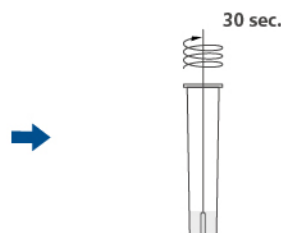

Insert the swab into the tube and swirl for 30 seconds.

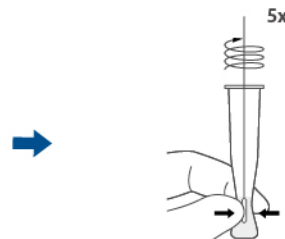


Rotate the swab 5 times while squeezing the side of the tube.

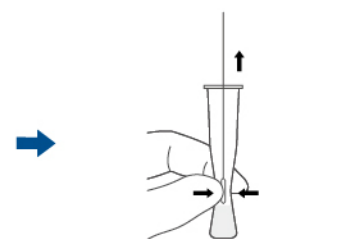

Remove the swab while squeezing the tube.

4. Running the Test

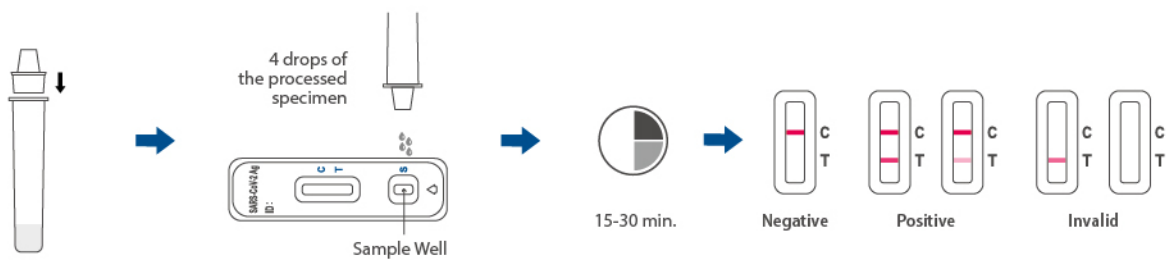

Ordering Information

Product Name	Catalog No.	Format	Specimen	Package
Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)	L031-118M5 ✓	Cassette	Anterior nasal swabs	1 Test/Kit
	L031-118P5 ✓	Cassette	Anterior nasal swabs	5 Tests/Kit
	L031-118R5 ✓	Cassette	Anterior nasal swabs	25 Tests/Kit

✓ CE Marked